

PACKAGE INSERT

**INTENDED USE**

The HIV-2 Mixed Titer AccuSet™ Performance Panel is intended for use by diagnostic manufacturers, researchers, and clinical laboratories to develop, evaluate, or troubleshoot anti-HIV-2, anti-HIV-1/2, and HIV Ag/Ab Combo test methods. Characterized samples and comprehensive data are provided for comparative analysis. For research use only. Not for use in diagnostic procedures.

**PRODUCT DESCRIPTION**

This product is a 9-member panel consisting of undiluted, naturally occurring plasma samples. Panel members represent bleeds from multiple individuals positive for HIV-2 with varying reactivity for anti-HIV-2. Each sample represents a single collection event. One sample is included as a non-reactive sample and is negative for all HIV methods tested. No preservatives were added.

Cat. No. 0800-0343      1 vial per member  
   9 members, 1.2 mL per vial

**STORAGE**

Panel members should be stored frozen at -70°C or colder. SeraCare Life Sciences recommends that the panel members be divided into smaller aliquots to avoid multiple freeze-thaw cycles if appropriate. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

**INTERPRETATION OF RESULTS**

The Data Sheet for the HIV-2 Mixed Titer AccuSet™ Performance Panel is available at [www.seracare.com](http://www.seracare.com). The Data Sheet lists results for panel members generated using commercially available screening, monitoring, and

confirmatory test methods. Tests were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet. Data Sheets are updated when new data are available.

**LIMITATIONS**

The HIV-2 Mixed Titer AccuSet™ Performance Panel is offered for research use only, not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

**PRECAUTIONS**

Follow Universal Precautions.<sup>1</sup> The units that make up this panel were tested for HBsAg and anti-HCV and were found negative. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

**REFERENCES**

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

**For assistance, contact SeraCare Technical Support at 508.244.6400.**

The Package Insert and Data Sheet for this panel in PDF form can be found at [www.seracare.com](http://www.seracare.com)

A printed copy of the Data Sheet may be requested by email at [info@seracare.com](mailto:info@seracare.com), or by phone at 508.244.6400

**ASK ABOUT RELATED SERACARE PRODUCTS**

- AccuType™ Viral Isolates
- ACCURUN® Quality Controls
- Disease State Biological Materials
- AccuVert™ Seroconversion Panels

