

Anti-HSV 1/2 Mixed Titer Performance Panel PTH202

PACKAGE INSERT

INTENDED USE

Anti-HSV 1/2 Mixed Titer Performance Panel PTH202 is a panel of naturally occurring plasma and serum specimens. PTH202 is intended for use by diagnostic manufacturers and clinical laboratories and researchers to develop, evaluate or troubleshoot their anti-HSV 1/2, anti-HSV-1, and anti-HSV 2 tests methods using well characterized specimens, and to provide comprehensive data for comparative analysis. For Research Use Only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

PTH202 consists of a set of 21 members with reactivity ranging from negative to strongly positive for anti-HSV 1/2 IgG and IgM markers. Specimens are undiluted aliquots from plasma and serum units. Units were aseptically filtered; no preservative was added.

Cat. No. PTH202-1.2 1 vial per member
 21 members, 1.2 mL per vial

STORAGE

Panel members should be stored frozen at -20°C or colder. SeraCare Life Sciences recommends that the panel members be divided into smaller aliquots to avoid multiple freeze-thaw cycles, if appropriate. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The Data Sheet for anti-HSV 1/2 Mixed Titer Performance Panel PTH202-1.2 is available at www.seracarepanels.com. Select PTH202 from the list of

panel data sheets. The Data Sheet lists results for panel members generated using commercially available screening and monitoring tests approved in the U.S. and/or the European Union. The tests that are listed were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet.

LIMITATIONS

PTH202 is offered for research use only. Not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

Follow Universal Precautions¹. The units that make up this panel were tested and found negative for HBsAg, anti-HIV 1/2, and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. CDC recommendations for prevention of HIV transmission in health care settings. MMWR 36 (supp.2) 1987.

For assistance, contact SeraCare Technical Support at 508.244.6400.

The Data Sheet for this panel in PDF form can be found at www.seracarepanels.com

The printed Package Insert or Data Sheet may be requested by email at info@seracare.com, or by phone at 508.244.6400

ASK ABOUT RELATED SERACARE PRODUCTS

- ACCURUN[®] independent quality controls
- SeraCon[™] and other processed plasma products
- Characterized Disease State plasma including a diverse inventory of patient samples and full units

