

Anti-*Toxoplasma gondii* Mixed Titer Performance Panel PTT202

PACKAGE INSERT

INTENDED USE

The Anti-*Toxoplasma gondii* Mixed Titer Performance Panel PTT202 is a panel of naturally occurring plasma specimens. PTT202 is intended for use by diagnostic manufacturers and clinical laboratories to evaluate *Toxoplasma* test systems using well characterized specimens and to provide comprehensive data for comparative analysis. For Research Use Only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

PTT202 consists of a set of 21 aliquots from plasma units with reactivity ranging from negative to strongly positive for anti-*T. gondii*. Units were aseptically filtered; no preservative was added.

Cat. No. PTT202-1.2 1 vial per member
 21 members, 1.2 mL per vial

STORAGE

Panel members should be stored frozen at -20°C or colder. SeraCare Life Sciences recommends that the panel members be divided into smaller aliquots for ease in repeated testing and to avoid multiple freeze-thaw cycles. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The Data Sheet for Anti-*Toxoplasma gondii* Mixed Titer Performance Panel PTT202 is available at www.seracarepanels.com. Select PTT202 from the list of panel data sheets. The Data Sheet lists results for panel members generated using commercially available

screening and monitoring tests approved in the U.S. and/or the European Union. The tests that are listed were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet. Data Sheets are updated when new data are available.

LIMITATIONS

PTT202 is offered for research use only. Not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

Follow Universal Precautions.¹ The units that make up this panel were tested and found negative for HBsAg, anti-HIV 1/2, and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. CDC recommendations for prevention of HIV transmission in health care settings. MMWR 36 (supp.2) 1987.

For assistance, contact SeraCare Technical Support at 508.244.6400.

ASK ABOUT RELATED SERACARE PRODUCTS

- ACCURUN® independent quality controls
- SeraCon™ and other processed plasma products
- Characterized Disease State plasma including a diverse inventory of patient samples and full units

The Data Sheet for this panel in PDF form can be found at www.seracarepanels.com

The printed Data Sheet may be requested by email at info@seracare.com, or by phone at 508.244.6400