

Anti-Hepatitis A Virus (HAV) Mixed Titer Performance Panel

PHT202

A SERACARE PANEL PRODUCT

INTENDED USE

The Anti-Hepatitis A Virus (HAV) Mixed Titer Performance Panel PHT202 is a panel of naturally occurring plasma specimens. This panel is intended for use by diagnostic manufacturers and clinical laboratories to evaluate their anti-Hepatitis A tests with characterized samples and to provide comprehensive data for comparative analysis. For Research Use Only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

PHT202 consists of a set of 21 aliquots from plasma units with reactivities ranging from negative to strongly positive for Hepatitis A IgG and/or IgM antibody. One specimen negative for Hepatitis A IgG and IgM antibody is included as a nonreactive sample. Specimens are undiluted aliquots from plasma units. Units were aseptically filtered and no preservative was added.

Cat. No. PHT202-1.2 1 vial per member
21 members, 1.2 mL per vial

STORAGE

Panel members should be stored frozen at -10°C or colder. SeraCare Life Sciences recommends that the panel members be divided into smaller aliquots for ease in repeated testing and to avoid multiple freeze-thaw cycles. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The Data Sheet for Anti-Hepatitis A Virus (HAV) Mixed Titer Performance Panel PHT202 is available at www.seracarepanels.com. Select PHT202 from the list

The Data Sheet for this panel in PDF form can be found at www.seracarepanels.com

The printed Data Sheet may be requested by email at info@seracare.com, or by phone at 508.244.6400

of panel data sheets. The Data Sheet lists results for panel members generated using commercially available screening and monitoring tests approved in the U.S. and/or the European Union. The tests that are listed were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet. Data Sheets are updated when new data are available.

LIMITATIONS

PHT202 is offered for research use only. Not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

These materials have not been treated and should be considered biohazardous. Follow Universal Precautions.¹ The units that make up this panel were tested and found negative for HBsAg, anti-HIV, and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. CDC recommendations for prevention of HIV transmission in health care settings. MMWR 36 (supp.2) 1987.

For assistance, contact SeraCare Technical Support at 508.244.6400.

ASK ABOUT RELATED SERACARE PRODUCTS

- ACCURUN® independent quality controls
- SeraCon™ and other processed plasma products
- Global Patient Sample (GPS) Program – access to a vast and evolving inventory of single test patient samples

