

HCV AccuVert™ Seroconversion Panel Modified PHV920(M) (0610-0218)

PACKAGE INSERT

INTENDED USE

The HCV AccuVert™ Seroconversion Panel Modified PHV920(M) is intended for use by diagnostic manufacturers, researchers, and clinical laboratories to develop, evaluate, or troubleshoot HCV test methods. Characterized samples and comprehensive data are provided for comparative analysis. For research use only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

This product is a 9-member HCV seroconversion panel. Panel members represent serial bleeds from a single donor during the development and progression of a HCV infection. Panel members consist of undiluted, naturally occurring plasma samples. Each sample represents a single collection event. No preservatives were added.

Cat. No. 0610-0218 1 vial per member
 9 members, 1.0 mL per vial

STORAGE

Panel members should be stored frozen at -70°C or colder. HCV antibody will be preserved at -10°C or colder. SeraCare recommends that the panel members be divided into smaller aliquots to avoid multiple freeze-thaw cycles, if appropriate. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The data sheet for HCV AccuVert™ Seroconversion Panel Modified PHV920(M) is available at www.seracare.com. The data sheet lists results for panel members generated using commercially-available screening, monitoring, and

confirmatory test methods. Tests were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the data sheet.

LIMITATIONS

HCV AccuVert™ Seroconversion Panel Modified PHV920(M) is offered for research use only, not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

Follow Universal Precautions.¹ The units that make up this panel were tested and found negative for anti-HIV and HBsAg. This does not ensure the absence of these or other human pathogens. Most panel members were found positive for HCV RNA and anti-HCV.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

For assistance, contact SeraCare at 508.244.6400.

The Package Insert and Data Sheet for this panel in PDF form can be found at www.seracare.com

A printed copy of the Data Sheet may be requested by email at info@seracare.com, or by phone at 508.244.6400

ASK ABOUT RELATED SERACARE PRODUCTS

- ACCURUN® Quality Controls
- AccuSet™ Performance Panels
- Disease State Biological Materials
- SeraCon™ and Basematrix Processed Plasma

