

# AccuVert™ Syphilis Seroconversion Panel PSS901 (0615-0017)

## INTENDED USE

AccuVert™ Syphilis Seroconversion Panel PSS901 (0615-0017) is a group of serial bleeds from a single individual during early Syphilis infection. This Panel is intended for use by diagnostics manufacturers and clinical laboratories and researchers to develop, evaluate or troubleshoot tests for Syphilis markers, and to study Syphilis infection. For Research Use Only. Not for use in diagnostic procedures.

## PRODUCT DESCRIPTION

This product consists of a set of nine undiluted, unpreserved plasma samples from a single plasma donor collected during a period of Syphilis Seroconversion. All units were maintained frozen, except for the interval of dispensing into vials. No preservatives were added.

Item No. PSS901-1.2    1 vial per member  
(0615-0017)        9 members, 1.2 mL per vial

## STORAGE

Panel members should be stored frozen at -20°C or colder. SeraCare recommends that the panel members be divided into smaller aliquots, if appropriate, to avoid multiple freeze-thaw cycles. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

## INTERPRETATION OF RESULTS

The Data Sheet for AccuVert Syphilis Seroconversion Panel is available at [www.seracare.com](http://www.seracare.com). The Data Sheet lists results for panel members generated using commercially available screening, monitoring and confirmatory tests approved in the U.S. and/or the European Union. Tests were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet. Data Sheets are updated when new data are available.

## LIMITATIONS

The AccuVert Syphilis Seroconversion Panel is offered for research use only. Not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

## PRECAUTIONS

Follow Universal Precautions<sup>1</sup>. Panel members were tested and found negative for HBsAg, anti-HCV and anti-HIV 1/2. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

## REFERENCES

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

**For assistance, contact SeraCare at 508.244.6400.**

The package insert for this panel can be found at [www.seracare.com](http://www.seracare.com)

A printed copy of the package insert or data sheet may be requested by email at [info@seracare.com](mailto:info@seracare.com) or by phone at 508.244.6400