

AccuSet™ HBsAg Mixed Titer Performance Panel Modified PHA207 (0805-0314)

INTENDED USE

The AccuSet™ HBsAg Mixed Titer Performance Panel Modified is intended for use by diagnostic manufacturers, researchers, and clinical laboratories to develop, evaluate, or troubleshoot HBsAg test methods. Characterized samples and comprehensive data are provided for comparative analysis. For research use only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

This product is a modified 12-member panel originating from HBsAg Mixed Titer Performance Panel PHA207-1.5. Member #4, 6, 10, 11, 12, 16, 17, 19 and 20 from the original 21-member panel are no longer available. Panel members are undiluted, naturally occurring plasma samples. Ten panel members are included that provide a broad range of reactivity for HBsAg. Two samples are included as non-reactive and negative by all methods tested. No preservatives were added.

Item No. 0805-0314 1 vial per member
 12 members, 1.5 mL per vial

STORAGE

Panel members should be stored at -20 °C or colder. SeraCare Life Sciences recommends that the panel members be divided into smaller aliquots to avoid multiple freeze-thaw cycles, if appropriate. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The Data Sheet for AccuSet HBsAg Mixed Titer Performance Panel Modified is available at www.seracare.com. The Data Sheet lists results for panel members generated using commercially-available screening, monitoring, and confirmatory test methods. Tests were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet. Data Sheets are updated when new data are available.

LIMITATIONS

The AccuSet HBsAg Mixed Titer Performance Panel Modified is offered for research use only, not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

Follow Universal Precautions¹. The units that make up this panel were tested and found negative for anti-HIV-1 and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled. These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

For assistance, contact SeraCare at 508.244.6400.

The package insert for this panel can be found at www.seracare.com

A printed copy of the package insert or data sheet may be requested by email at info@seracare.com or by phone at 508.244.6400