

Anti-*Toxoplasma gondii* Mixed Titer AccuSet™ Performance Panel 0815-0004 (PTT202)

INTENDED USE

The Anti-*Toxoplasma gondii* Mixed Titer AccuSet™ Performance Panel 0815-0004 (PTT202) is a panel of naturally occurring plasma specimens. This panel is intended for use by diagnostic manufacturers and clinical laboratories to evaluate *Toxoplasma* test systems using well characterized specimens and to provide comprehensive data for comparative analysis. For Research Use Only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

This product consists of a set of 21 aliquots from plasma units with reactivity ranging from negative to strongly positive for anti-*T. gondii*. Units were aseptically filtered; no preservative was added.

Item No. 0815-0004 1 vial per member
 21 members, 1.2 mL per vial

STORAGE

Panel members should be stored frozen at -20 °C or colder. SeraCare Life Sciences recommends that the panel members be divided into smaller aliquots for ease in repeated testing and to avoid multiple freeze-thaw cycles. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The data sheet for Anti-*Toxoplasma gondii* Mixed Titer AccuSet Performance Panel 0815-0004 (PTT202) is available at www.seracare.com. The data sheet lists results for panel members generated using commercially available screening and monitoring tests approved in the U.S. and/or the European Union. The tests that are listed were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the data sheet. Data Sheets are updated when new data are available.

LIMITATIONS

Anti-*Toxoplasma gondii* Mixed Titer AccuSet Performance Panel 0815-0004 (PTT202) is offered for research use only. Not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

Follow Universal Precautions¹. The units that make up this panel were tested and found negative for HBsAg, anti-HIV 1/2, and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

For assistance, contact SeraCare at 508.244.6400.

The package insert for this panel can be found at www.seracare.com

A printed copy of the package insert or data sheet may be requested by email at info@seracare.com or by phone at 508.244.6400