

# AccuSet™ Anti-Hepatitis A Virus (HAV) Mixed Titer Performance Panel 0820-0443

## INTENDED USE

The AccuSet™ anti-HAV Mixed Titer Performance Panel 0820-0443 is a panel of naturally occurring plasma specimens. This panel is intended for use by diagnostics manufacturers, researchers, and clinical laboratories to develop, evaluate, and troubleshoot HAV test methods. Characterized samples and comprehensive data are provided for comparative analysis. For Research Use Only. Not for use in diagnostic procedures.

## PRODUCT DESCRIPTION

This product is a 21-member panel consisting of undiluted, naturally occurring plasma samples collected from individual donors for HAV infection with mixed reactivities on commercially available anti-HAV assays. Each sample represents a single collection event. No preservatives were added.

Material Number: 0820-0443, 1 vial per member  
21 members, 1.2 mL per vial

## STORAGE

Panel members should be stored frozen at -20°C or colder. LGC SeraCare recommends that the panel members be divided into smaller aliquots, if appropriate, to avoid multiple freeze-thaw cycles. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

## INTERPRETATION OF RESULTS

The Data Sheet for AccuSet anti-Hepatitis A Virus (HAV) Mixed Titer Performance Panel 0820-0443 is available at [www.seracare.com](http://www.seracare.com). The Data Sheet lists results for panel members generated using commercially available screening, monitoring and confirmatory tests approved in the U.S. and/or the European Union. The tests that are listed were performed at LGC SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the Data Sheet. Data Sheets are updated when new data are available.

## LIMITATIONS

AccuSet anti-HAV Mixed Titer Performance Panel 0820-0443 is offered for research use only. Not for use in diagnostic procedures. Data are provided for informational purposes. LGC SeraCare Life Sciences does not claim that others can duplicate test results exactly.

## SAFETY PRECAUTIONS

These materials have not been treated and should be considered biohazardous. Follow Universal Precautions<sup>1</sup>. The units that make up this panel were tested and found negative for HBsAg, anti-HIV, and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

## REFERENCES

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

**For assistance, contact LGC SeraCare at  
508.244.6400.**

The package insert for this panel can be found at [www.seracare.com](http://www.seracare.com).

A printed copy of the package insert or data sheet may be requested by email at [CDx-Info@LGCGroup.com](mailto:CDx-Info@LGCGroup.com) or by phone at 508.244.6400.