

AccuSet™ Zika Performance Panel

0845-0142

INTENDED USE

The AccuSet™ Zika Performance Panel (0845-0142) is intended for use by diagnostic manufacturers, researchers, and clinical laboratories to develop, evaluate, or troubleshoot Zika test methods. Characterized samples and comprehensive data are provided for comparative analysis. For research use only. Not for use in diagnostic procedures.

PRODUCT DESCRIPTION

This product is a 10-member performance panel. Panel members are undiluted, naturally occurring plasma samples with reactivity ranging from negative to positive for Zika IgM antibodies. Each sample represents a single collection event. No preservatives were added.

Item No. 0845-0142 1 vial per member
 10 members, 1.0 mL per vial

STORAGE

Panel members should be stored at -20 °C or colder. SeraCare recommends that the panel members be divided into smaller aliquots to avoid multiple freeze-thaw cycles, if appropriate. If turbidity or particulate matter is observed, the samples should be centrifuged in accordance with each test kit manufacturer's instructions for sample preparation.

INTERPRETATION OF RESULTS

The data sheet for the AccuSet Zika Performance Panel (0845-0142) is available at www.seracare.com. The data sheet lists results for panel members generated using commercially-available test methods. Tests were performed at SeraCare or at recognized reference laboratories (RL) by individuals who routinely use these procedures. Information regarding specific test methods is available on the data sheet.

LIMITATIONS

The AccuSet Zika Performance Panel is offered for research use only, not for use in diagnostic procedures. Data are provided for informational purposes. SeraCare Life Sciences does not claim that others can duplicate test results exactly.

PRECAUTIONS

Follow Universal Precautions¹. The units that make up this panel were tested and found negative for anti-HIV 1/2, HBsAg and anti-HCV. This does not ensure the absence of these or other human pathogens.

Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

These materials should be disposed of in a manner that will inactivate pathogenic agents.

REFERENCES

1. Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.

For assistance, contact SeraCare at 508.244.6400.

The package insert for this panel can be found at www.seracare.com

A printed copy of the package insert or data sheet may be requested by email at info@seracare.com or by phone at 508.244.6400