


# ACCURUN® 800

## ToRCH Negative Control

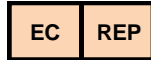

LGC Clinical Diagnostics, Inc. | 37 Birch Street, Milford, MA 01757 USA  
Phone: +1 508.244.6400 | CDx-Info@LGCGroup.com

MEDIMARK® Europe  
11, rue Émile Zola BP 2332  
38033 Grenoble Cedex 2 – France  
+ 33 (0) 4 76 86 43 22  
info@medimark-europe.com

10025GB-21 September 2021

## Explanation of symbols used in LGC Clinical Diagnostics product labeling



Upper limit of temperature



Temperature limitation



Authorized Representative in  
the European Community



Biological risks



Use By



*In Vitro* Diagnostic Medical Device



Negative control



Catalogue number



Consult instructions for use



Positive control



Batch code



Manufacturer



Control



Highly Flammable



Toxic by inhalation, in contact  
with skin and if swallowed



Health Hazard



# ACCURUN® 800 ToRCH Negative Control

## NAME AND INTENDED USE

ACCURUN controls are intended to estimate laboratory testing precision and can be used to detect errors in laboratory testing procedures. ACCURUN® 800 ToRCH Negative Control is formulated for use with *in vitro* diagnostic test kits for the detection of IgG and IgM antibodies to *Toxoplasma gondii*, Rubella virus, Cytomegalovirus, and Herpes Simplex virus. This product is not intended for use in testing blood or plasma donors. Positive controls for many of these analytes are available separately from LGC Clinical Diagnostics. *For In Vitro Diagnostic Use.*

## SUMMARY

Frequent testing of independent quality control samples provides the analyst with a means of monitoring the performance of laboratory assays. Routine use of controls enables laboratories to monitor day-to-day test variation, lot-to-lot performance of test kits, and operator variation, and can assist in identifying increases in random or systematic error. A well designed quality control program can provide added confidence in the reliability of results obtained for unknown specimens. The use of non-reactive samples as independent controls may provide valuable information concerning laboratory proficiency and kit lot variation that may affect assay sensitivity<sup>1</sup>.

## PRINCIPLES OF THE PROCEDURE

ACCURUN 800 ToRCH Negative Control is designed for use with *in vitro* assay procedures for purposes of monitoring test performance. ACCURUN 800 ToRCH Negative Control is manufactured from human serum or plasma nonreactive for HBsAg and antibodies to *Toxoplasma gondii*, Rubella virus, Cytomegalovirus, Herpes Simplex virus, HIV 1 and 2, HTLV, and HCV. ACCURUN controls do not have assigned values. This control is formulated to produce negative reactivity in those manufacturers' assays listed in Table 1. Specific levels of reactivity will vary among different manufacturers' assays, different procedures, different lot numbers, and different laboratories.

## REAGENTS

Item No. 2010-0018 1 vial, 1.0 ml per vial

This control contains stabilizers (EDTA, buffering agents), and 0.1% ProClin® (5-chloro-2-methyl-4-isothiazolin-3-one&2-methyl-4-isothiazolin-3-one) as preservative.

## WARNINGS AND PRECAUTIONS

### For In Vitro Diagnostic Use

CAUTION: Handle ACCURUN controls and all human blood products as though capable of transmitting infectious agents. ACCURUN 800 ToRCH Negative Control is manufactured from human serum or plasma nonreactive for HBsAg, and antibodies to HIV 1 and 2, HTLV, and HCV with current FDA licensed tests.

### Safety Precautions

Use the Centers for Disease Control (CDC) recommended universal precautions for handling ACCURUN and human blood<sup>2</sup>. Do not pipette by mouth; do not eat or drink in areas where specimens are being handled. Clean any spillage by immediately wiping up with 0.5% sodium hypochlorite solution. Dispose of all specimens, controls, and materials used in testing as though they contain infectious agents.

### Handling Precautions

Do not use ACCURUN controls beyond the expiration date. Avoid microbial contamination of the controls when opening and closing the vials.

## STORAGE INSTRUCTIONS

Store ACCURUN 800 ToRCH Negative Control at 2-8°C. Once opened, return control to refrigerated storage immediately after use, and discard after 60 days. After opening, record the date opened and the expiration date on the vial. Multiple freeze-thaw cycles are not recommended, and may have variable adverse effects on test results. To prevent leakage, store vials upright.

## INDICATIONS OF REAGENT INSTABILITY OR DETERIORATION

Alterations in physical appearance may indicate instability or deterioration of ACCURUN controls. Solutions that are visibly turbid should be discarded.

## PROCEDURE

### Materials Provided

ACCURUN 800 ToRCH Negative Control is manufactured from human serum or plasma nonreactive for HBsAg, and antibodies to *Toxoplasma gondii*, Rubella virus, Cytomegalovirus, Herpes Simplex virus, HIV 1 and 2, HTLV, and HCV.

### Materials Required but not Provided

Refer to instructions supplied by manufacturers of the test kits to be used.

### Instructions for Use

Allow the controls to reach room temperature prior to use, then return controls to refrigerated storage immediately after use. Mix the contents of the vials by gently swirling. ACCURUN controls should be included in a test run using exactly the same procedure provided by the manufacturer for unknown specimens. ACCURUN controls must NOT be substituted for the positive and negative control reagents provided with manufactured test kits.

## Quality Control

Since ACCURUN controls do not have assigned values, it is recommended that each laboratory validate the use of each lot of ACCURUN with each specific assay system prior to its routine use in the laboratory.

## INTERPRETATION OF RESULTS

Levels of reactivity of ACCURUN 800 ToRCH Negative Control may vary with different manufacturers' tests and different test kit lots. Since the control does not have an assigned value, the laboratory must establish a range for each lot of ACCURUN 800 ToRCH Negative Control. When results for ACCURUN 800 ToRCH Negative Control are outside the established acceptable range of values, it may be an indication of unsatisfactory test performance. Possible sources of discrepancy include: deterioration of test kit reagents, operator error, faulty performance of equipment, or contamination of reagents.

## LIMITATIONS OF THE PROCEDURE

ACCURUN CONTROLS MUST NOT BE SUBSTITUTED FOR THE POSITIVE AND NEGATIVE CONTROL REAGENTS PROVIDED WITH MANUFACTURED TEST KITS.

TEST PROCEDURES and INTERPRETATION OF RESULTS provided by manufacturers of test kits must be followed closely. Deviations from procedures recommended by test kit manufacturers may produce unreliable results. ACCURUN controls are not calibrators and should not be used for assay calibration. Performance characteristics for ACCURUN 800 ToRCH Negative Control have been established only for *Toxoplasma gondii*, Rubella virus, Cytomegalovirus and Herpes Simplex virus. Adverse shipping and/or storage conditions or use of outdated controls may produce erroneous results.

## EXPECTED RESULTS

ACCURUN 800 ToRCH Negative Control DOES NOT HAVE ASSIGNED VALUES. This control is formulated to produce negative reactivity in those manufacturers' assays listed in Table 1. Specific levels of reactivity will vary among different manufacturers' assays, different procedures, different lot numbers, and different laboratories. Procedures for implementing a quality assurance program and monitoring test performance on a routine basis must be established by each individual laboratory. Each laboratory should establish its own range of acceptable values for each analyte. For example, the acceptable range might include all values within 2 standard deviations of the mean of 20 data points obtained in 20 runs over a period of 30 days<sup>3</sup>.

## SPECIFIC PERFORMANCE CHARACTERISTICS

ACCURUN controls are designed for use with *in vitro* assay procedures for purposes of monitoring assay performance. ACCURUN 800 ToRCH Negative Control is manufactured from human serum or plasma nonreactive for HBsAg and antibodies to *Toxoplasma gondii*, Rubella virus, Cytomegalovirus and Herpes Simplex virus, HIV 1 and 2, HTLV, and HCV. ACCURUN controls do not have assigned values. This negative control is formulated to produce negative reactivity in those manufacturers' assays listed in Table 1. Additional data are available on request for other manufacturers' test kits. Specific levels of reactivity will vary among different manufacturers' assays, different procedures, different lot numbers, and different laboratories. Procedures for implementing a quality assurance program and monitoring test performance on a routine basis must be established by each individual laboratory.

## REFERENCES

- Green IV GA, Carey RN, Westgard JO, Carten T, Shablesky LA, Achord D, Page E, and Le AV. Quality control for qualitative assays: quantitative QC procedure designed to assure analytical quality required for an ELISA for hepatitis B surface antigen. Clin. Chem. 43:9 1618-1621, 1997.
- Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission of Infectious Agents in Healthcare Settings.
- Statistical Quality Control for Quantitative Measurements: Principles and Definitions. NCCLS document C24-A2, 1999.

Table 1. ACCURUN 800 ToRCH Negative Control is nonreactive in the following manufacturers' tests.

Analyte	Manufacturer	Product Name
Toxoplasma IgG	Trinity Biotech Jamestown, NY 14701	Captia™ Toxo IgG
Toxoplasma IgM	bioMérieux SA Marcy l'Etoile, France	VIDAS® Toxo IgM
Rubella IgG	Trinity Biotech Jamestown, NY 14701	Captia™ Rubella IgG
Rubella IgM	Trinity Biotech Jamestown, NY 14701	Captia™ Rubella IgM
CMV IgG	Trinity Biotech Jamestown, NY 14701	Captia™ CMV IgG
CMV IgM	Trinity Biotech Jamestown, NY 14701	Captia™ CMV IgM
HSV IgG	Trinity Biotech Jamestown, NY 14701	Captia™ Herpes Group IgG ELISA
HSV IgM	Wampole Laboratories Cranberry, NJ 08512	HSV 1/2 IgM II Assay

For assistance, contact LGC Clinical Diagnostics Technical Support at +1 508.244.6400.