

Representative Reactivities Table



**CLINICAL
DIAGNOSTICS**

Optitrol Chagas-1 (Positive Control)

Version: 1

Date: Apr 29, 2025

Analyte	Manufacturer	Assay Name	Method	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Anti-Trypanosoma cruzi IgG	Abbott	Architect Chagas	CMIA	S/CO	5.90	0.015	10	1	positive
Anti-Trypanosoma cruzi IgG	DiaSorin	LIAISON® XL murex Chagas	CLIA	S/CO	2.75	0.12	10	1	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.