



Representative Reactivities Table

Optitrol Yellow

Positive Control

Version: 1

Date: Jun 10, 2025

Analyte	Manufacturer	Assay Name	Method	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Anti-HCV IgG	Abbott	Alinity s Anti-HCV II Reagent Kit	CMIA	S/CO	2.66	0.03	10	1	positive
Anti-HIV 1 IgG	Abbott	Alinity s HIV Ag/Ab Combo Reagent Kit	CMIA	S/CO	5.57	0.13	10	1	positive
Anti-HTLV I IgG	Abbott	Alinity s HTLV-I/II Reagent Kit	CMIA	S/CO	3.58	0.08	10	1	positive
Anti-Treponema pallidum IgG	Abbott	Alinity s Syphilis Reagent Kit	CMIA	S/CO	2.80	0.03	10	1	positive
HBsAg	Abbott	Alinity s HBsAg Reagent Kit	CMIA	S/CO	3.03	0.05	10	1	positive
Anti-HBc IgG	Abbott	Alinity s Anti-HBc Reagent Kit	CMIA	S/CO	2.24	0.05	10	1	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values.
Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.