

KOVA-Trol® II Low Abnormal Value Sheet

Product Description: KOVA-Trol II Low Abnormal

Material Number: Kit 87130 (4 x 15 mL)

The representative ranges are determined statistically based on historical data and periodically verified with representative batches tested on the specified method.

Siemens Urinalysis Strip Testing							McKesson
Analyte	Multistix Visual	Clinitek 50	Clinitek 100	Clinitek Status/Status +	Clinitek Advantus	Clinitek Novus	
pH	7.0 - 8.5	7.0 - 8.5	7.5 - 8.5	7.0 - ≥9.0	6.5 - 8.5	7.0 - 8.5	7.0 - 8.5
Protein	Negative - 100	Negative - 30 mg/dL	Negative - 30 mg/dL	Negative - 30 mg/dL	Negative - 30 mg/dL	Negative - 30 mg/dL	Negative - 30 mg/dL
Glucose	100 - 500mg/dL	100 - 500 mg/dL	100 - 500 mg/dL	100 - 500 mg/dL	100 - 500 mg/dL	100 - 500 mg/dL	100 - 500 mg/dL
Ketones	5 - 40 mg/dL	Trace - 40 mg/dL	Trace - 40 mg/dL	Trace - 40 mg/dL	Trace - 40 mg/dL	Trace - 40 mg/dL	5 - 40 mg/dL
Bilirubin	1 + - 3 +	Moderate - Large	Small - Large	Moderate - Large	Small - Large	Small - Large	Negative - 1
Blood (hemoglobin)	1 + - 3 +	Trace-Lysed - Moderate	Trace - Moderate	Trace-Lysed - Moderate	Small - Moderate	Trace - Moderate	25 - 200 Ery/μL
Nitrite	Positive	Positive	Positive	Positive	Positive	Positive	Positive
Urobilinogen	0.2 - 1 EU/dL	0.2 - 1.0 EU/dL	0.2 - 1.0 EU/dL	0.2 - 1.0 EU/dL	0.2 - 1.0 EU/dL	0.2 - 1.0 EU/dL	0.2 - 1 mg/dL
Specific Gravity	1.005 - 1.020	≤1.005 - 1.015	≤1.005 - 1.025	≤1.005 - 1.025	≤1.005 - 1.020	≤1.005 - 1.015	1.005 - 1.020
Leukocyte Esterase	1 + - 3 +	Trace - Large	Trace - Large	Trace - Large	Trace - Large	Trace - Large	70 - 500 Leu/μL
Microalbumin	N/A	30 - 150 mg/L	30 - 150 mg/L	30 - 150 mg/L	N/A	10 - 150	N/A
Creatinine	50 - 300 mg/dL	100 - 300 mg/dL	100 - 300 mg/dL	100 - 300 mg/dL	100 - 300 mg/dL	100 - 300 mg/dL	N/A
A : C	N/A	< 30 Normal - 30 - 300 mg/g Abnormal			N/A	Normal	N/A

KOVA-Trol® II Low Abnormal Value Sheet

Product Description: KOVA-Trol II Low Abnormal

Material Number: Kit 87130 (4 x 15 mL)

The representative ranges are determined statistically based on historical data and periodically verified with representative batches tested on the specified method.

Roche Urinalysis Strip Testing							
Analyte	Combur Visual	Chemstrip Visual	Chemstrip Criterion II	Urisys 1800	Urisys 1100	Cobas U411	Cobas U601
pH	7-8	7-8	7-8	7-8	7-8	7-8	7-8
Protein	Negative - 30 mg/dL	Negative - 30 mg/dL	Negative - 30 mg/dL	Negative - 25 mg/dL	Negative - 30 mg/dL	Negative - 75 mg/dL	Negative - 30 mg/dL
Glucose	100 - 300 mg/dL	100 - 500 mg/dL	100 - 250 mg/dL	100 - 300 mg/dL	100 - 250 mg/dL	100 - 300 mg/dL	100 - 250 mg/dL
Ketones	1 + - 2 +	1 + - 2 +	5 - 50 mg/dL	5 - 50 mg/dL	5 - 50 mg/dL	5 - 50 mg/dL	5 - 50 mg/dL
Bilirubin	1 + - 2+	1 + - 2 +	Negative - 1 mg/dL	1 - 3 mg/dL	1 - 3 mg/dL	1 - 3 mg/dL	1 - 3 mg/dL
Blood (hemoglobin)	5-10 - 250 Ery/μL	Trace - 250 Ery/μL	50 - 250 Ery/μL	50 - 150 Ery/μL	50 - 250 Ery/μL	50 - 250 Ery/μL	50 - 250 Ery/μL
Nitrite	Positive	Positive	Positive	Positive	Positive	Positive	Positive
Urobilinogen	Normal - 1 mg/dL	Normal - 1 mg/dL	Normal - 1 mg/dL	Normal - 1 mg/dL	Normal - 1 mg/dL	Normal - 1 mg/dL	Normal - 1 mg/dL
Specific Gravity	1.000 - 1.010	1.000 - 1.010	1.005 - 1.015	1.005 - 1.015	1.005 - 1.015	1.005 - 1.015	1.000 - 1.015
Leukocyte Esterase	10 - 25 - 500 Leu/μL	Trace - 2+	25 - 500 Leu/μL	25 - 500 Leu/μL	25 - 500 Leu/μL	100 - 500 Leu/μL	25 - 500 Leu/μL

KOVA-Trol® II Low Abnormal Value Sheet

Product Description: KOVA-Trol II Low Abnormal
Material Number: Kit 87130 (4 x 15 mL)

The representative ranges presented below are confirmed for each batch.

Physical and Microscopic Characteristics		
Marker	Method	Range
Specific Gravity	Refractometer (digital)	≤1.005 - 1.025
Osmolality	Osmometer	100 – 400 mOsm/kg
pH	pH Meter	7.0 – 8.5
Appearance	Visual	Clear – Slightly Cloudy
Color	Visual	Yellow - Green
RBC Mean Cell Count	Microscopy	1 – 12 cells/hpf*
WBC Mean Cell Count	Microscopy	1 – 12 cells/hpf*
Crystals	Microscopy	Present
Casts	Microscopy	<1

* Cells per high power field as measured using 10X eyepiece/40X objective

Pregnancy Testing		
Marker	Method	Range
hCG	Quidel QuickVue hCG Urine Test	Negative